

clinell®

Surface Disinfection



Clinell Universal Wipes kill COVID-19 in just 30 seconds.

A 2-in-1 combined cleaning and disinfectant, Clinell Universal Wipes are the No.1 disinfectant wipe in Australian Hospitals, killing at least 99.99% of germs.

An easy to use product with proven efficacy, to help keep you safe moving while working out on the road.



When to use

Clean these frequently touched surfaces at the beginning and end of your shift.




Windows, window controls & handles



Seats & seat belt clips



Heat & air conditioner controls

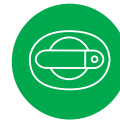

External & internal door & boot handles



Visors & centre console



Navigation & radio controls



Steering wheel



Seats



Gear stick & handbrake




Rear view mirror

How to use


Please follow the steps below:




Take one Clinell Universal Wipe from pack, ensuring you close the pack lid.

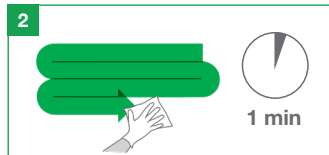

Wipe over high touch surfaces using friction and pressure, and in an S shape pattern. Allow the surface to air dry for one minute.

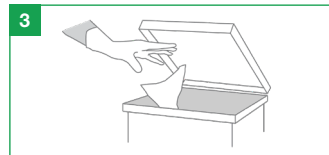

Remove gloves, place used gloves and wipes in rubbish bag.

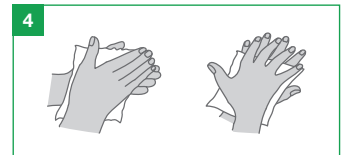

Wash your hands thoroughly with soap and water or by using hand wipes.

GHA210440