

Detergent Wipes

Instructions for use



For multi-surface general cleaning and damp dusting.

To clean non-infectious hard surfaces and equipment.

Not to be used on invasive equipment. Ensure lid is closed after each use to prevent wipes from drying out.



Product Code: **CDW215AUS**

Range of uses

(Examples of use - this is not an exhaustive list).

Surfaces

- Chairs
- Desks
- Door handles
- Drip stands
- Lockers
- Worktops
- Keyboards & accessories
- Trolleys

Equipment

- Blood pressure cuffs/machines
- ANTT (blue tray)
- Routine commode cleaning
- Walking frames
- Wheelchairs
- Pulse oximeter probes
- Hoists

Instructions for use




Risk assess to ensure appropriate PPE is worn, as per local protocol.



Remove one wipe from the pack.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.

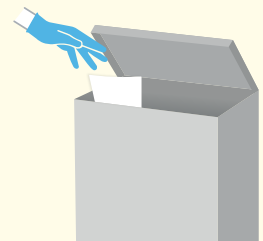

Dispose of as per local protocol.

Always refer to manufacturer's guidelines for compatibility with specialist equipment. Do not flush.
Always read the label and product information before use.