

Detergent Wipes

Stronger cleaning power,
ultra-low smear



Detergent Wipes

Cutting edge technology combined with ease of use for an incredibly effective formula.

Patented formula

Safe formulation for an ultra-low smear clean.

Optimum cleaning

Even stronger cleaning power, developed specifically for use in healthcare environments.

Convenient and effective

Disposable wipes that are ideal for general cleaning and damp-dusting.

Multi-surface cleaning

Can be used on both surfaces and equipment.

Range of uses

(Examples of use - this is not an exhaustive list).

Surfaces

- Mattresses
- Bed frames
- Table tops
- Desks
- Chairs
- Ledges
- Examination couches

Equipment

- Telephones
- Keyboards
- Accessories
- Wheelchairs
- Trolleys
- Walking frames

Optimum cleaning

Multi-surface detergent wipes for a superior level of cleaning and damp dusting.

Clinell Detergent Wipes are unlike any other. Our unique formula is incredibly effective at providing optimum multi-surface cleaning and damp dusting within healthcare settings. Many other detergent products are oil and phosphate based, which leaves smears and residue.

Our patented formula includes compounds that lower the surface tension of a liquid and act as a detergent. The surfactant molecules are designed for ultra-low smearing and excellent cleaning power.

Containing no alcohol or disinfectant, Clinell Detergent Wipes are extremely safe to use and are derived from sustainable plant sources.


How to use Clinell Detergent Wipes




Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol)



Remove one wipe from the pack

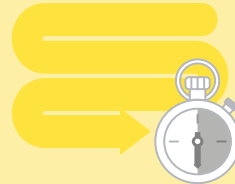

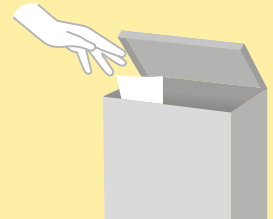

Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern. Let the surface air dry



Should your wipe dry out or become soiled, replace it with a new wipe

Order information

	Product	Unit of issue	Product code
	Detergent Wipes 215	Pack of 215	CDW215AUS
	White Wall Mounted Dispenser	(For CDW215AUS)	CDWDWAUS

For more information, speak to your GAMA Healthcare Sales Manager or visit www.gamahealthcare.com.au.
Always follow medical equipment manufacturer's cleaning procedures and guidelines.